

## Double walled Na Citrate coagulation and PTT test tubes

Polyethylene terephthalate (PET) tubes.

Resistant to mechanical stress, centrifugation, breakage in normal conditions of use and in case of accidental falls.

Polyethylene terephthalate (PET) is a transparent material that allows visual inspection of blood and biological fluids collected.



STERILE

Sterile

The closure system consists of a pierceable butyl rubber stopper and a polyethylene protection / safety cap.

Plastic label

APPLICATIONS: coagulation

Pack of 100 pieces - Packing 12 packs of 100 pieces

### Technical data

The tubes are manufactured according to the European standards ISO 6710 and EN 14820 "Disposable containers for the collection of human venous blood samples", as well as according to the CLSI guidelines relating to the manufacture of test tubes with predetermined vacuum and related devices for collection.

The sterilization processes are validated in compliance with the harmonized technical standards relating to Medical Devices.

The tubes are CE marked according to the IVD Directive 98/79 / EC "in vitro diagnostic medical devices", now amended.

CE Declarations of Conformity are available for IVD devices.

#### Recommendations for use

**Directions for mixing:** immediately after collection, shake the sample 4-5 times by slow inversion

**Minimum time before centrifugation:** none

**Maximum time before centrifugation:** 2 - 4 h after collection, depending on the test to be performed

**Centrifugation speed:** 2000-2500 g for 15-20 minutes at 20-25 C. Use centrifugation times and speeds to ensure a platelet-poor plasma

**Sample storage:** storage time and temperature may vary depending on the test to be performed

Code	Description	Diam. X height mm	Empty in ml	Cap color	Quantity of Na Citrate in ml	Percentage NA Citrate	Shelf life months	Pieces per pack.
2842400	Test tubes with Na Citrate 3.2%, mm 13x75 ml 0.9 blue cap	13x75	0.9	LIGHT BLUE	0.20	3.2	12	100

Code	Description	Diam. X height mm	Empty in ml	Cap color	Quantity of Na Citrate in ml	Percentag e NA Citrate	Shelf life months	Pieces per pack.
2842401	Test tubes with Na Citrate 3.8%, mm 13x75 ml 0.9 blue cap	13x75	0.9	LIGHT BLUE	0.20	3.8	12	100
2842450	Test tubes with Na Citrate 3.2%, mm 13x75 ml 1.8 blue cap	13x75	1.8	LIGHT BLUE	0.35	3.2	12	100
2842451	Test tubes with Na Citrate 3.2%, mm 13x75 ml 2.7 blue cap	13x75	2.7	LIGHT BLUE	0.35	3.2	12	100
2842490	Test tubes with Na Citrate 3.8%, mm 13x75 ml 1.8 blue cap	13x75	1.8	LIGHT BLUE	0.35	3.8	12	100
2842491	Test tubes with Na Citrate 3.8%, mm 13x75 ml 2.7 blue cap	13x75	2.7	LIGHT BLUE	0.35	3.8	12	100